

Adapter Instructions

ISI 189002 / 171028 For use with ISI Viking/Magnum

Please follow instructions when using ISI adapter 189002 and 171028 with the ISI Viking or Magnum with the OHD fit test adapter 9513-0139, Kit #1.

The Inhalation valve must be removed from the ISI part #171028. This is the blue diaphragm that is attached to the underside of the cross-hair plastic.

The 171028 must be attached to the front of the respirator. First you will remove the front portion of the respirator by unscrewing the 4 screws. The 171028 will replace the front portion of the respirator.

After mount has been secured, you will insert 189002 and turn clockwise until it is finger-tight.

Finally, you will insert and screw in the Kit #1 into the 189002.



Öã dā c a Á
0E0/Q c i} aā } a/Q &
ÚUÁO[c Á JI ÁO ^ T [c Á O Á î H F €
ì € È Ì G È H Á Á G F J È Ì Ì È Ì G Á Á c Á G F J È Ì Ì È Ì G
• a • O a & a d È { Á Á , , È & a d È { Á